

HESS Pole Mount Accessory

Healthcare Education Simulation Station

Revised 7/1/2025

www.BetterNurseEducation.com

PLEASE READ

DISCLAIMER

The information in the HESS is not intended or implied to be a substitute for professional medical expertise, advice, diagnosis or treatment.

There is no representation and no responsibility for the accuracy of information contained within the HESS.

The HESS is only intended to be used as an instructional aide by qualified medical educational professionals.

HESS Tablet Bracket Usage

Option 1 – Pole Mount

Pole Mount



Step 1
Tablet Face Down
Velcro Square In Center



Step 2
Clear Bracket Posts Up
Corners Posts Up



Step 3
Rubber Bands On Posts
Large Tablet - Doubled Bands
Small Tablet - 2 Small Bands Each



Step 4
Pole Clamp
Attached To Face Plate



Step 5
Face Plate Bands
Pulled Over Posts

Please be very careful separating the tablet and clamp from the clear bracket face plate when you are done. Start with one edge - and go slowly - to avoid pulling the Velcro pieces off the tablet, pole clamp or clear bracket face plate.

Option 2 – Equipment Mount

Equipment Mount




Step 1
Tablet Face Down
Velcro Square In Center



Step 2
Velcro Bands
"Hook" Side Up



Step 3
Clear Bracket Posts Up
Corners Posts Up

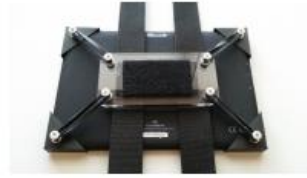

Step 4
Rubber Bands On Posts
Large Tablet - Doubled Bands
Small Tablet - 2 Small Bands Each

Please be very careful separating the tablet and clamp from the clear bracket face plate when you are done. Start with one edge - and go slowly - to avoid pulling the Velcro pieces off the tablet, pole clamp or clear bracket face plate.